# Singa<sup>®</sup>

## Diaphragm

- 1 **DE** Gebrauchsanweisung
- 9 **EN** Instruction manual
- 17 **FR** Mode d'emploi
- 25 **IT** Istruzioni per l'uso
- 33 **NL** Gebruiksaanwijzing
- 41 **DA** Brugsanvisning
- 49 **FI** Käyttöopas
- 57 **NO** Bruksanvisning
- 65 **SV** Bruksanvisning

Medintim

# Singa®

Diaphragm DE – deutsch

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### Diaphragma zur Kontrazeption

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# Singa®

Diaphragm EN – english

## Instruction manual Contraceptive diaphragm

Read the instructions for use before using the Singa® diaphragm.

The Singa® diaphragm comes in the following sizes: 60 mm, 65 mm, 70 mm, 75 mm, 80 mm, 85 mm and 90 mm diameter.


Before using the diaphragm for the first time, make an appointment with your doctor or family planning clinic to determine which size you need.

Your Singa® diaphragm is a safe and easy-to-use contraceptive.

**NOTE:** The Singa® diaphragm must always be used together with a diaphragm gel (e. g. Caya® Gel or Contragel®).

When used together with a diaphragm gel (e. g. Caya® Gel, Contragel® green), the Singa® diaphragm provides reliable protection against unwanted pregnancy.


Your Singa® diaphragm looks like this: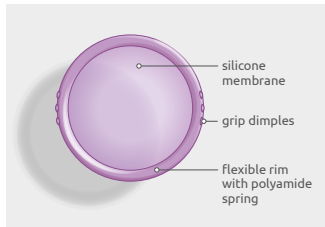

- **Silicone membrane:** covers the cervix to prevent sperm from reaching the uterus. Here you can see also the size of your diaphragm.
- **Grip dimples:** to hold and fold the diaphragm to ease insertion.
- **Flexible rim:** to ensure correct positioning.

## What is the position of the Singa® diaphragm in the vagina?


It completely covers the cervix in the upper vault of the vagina.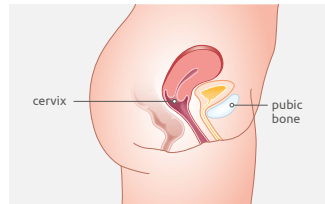


The Singa® diaphragm prevents sperm from reaching the uterus both during and after sexual intercourse.

## Before using Singa® the first time:

- Before using the Singa® diaphragm for the first time, wash it with soap and warm water, and dry it with a soft, clean cloth.
- Please make a note of the day of the first use on the back side of the case of the Singa® diaphragm.
- Before using the diaphragm for the first time, practise inserting it beforehand.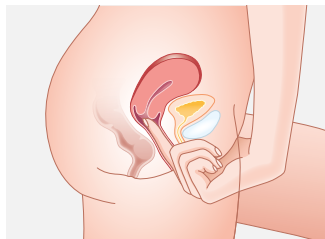

- Palpate your cervix – **Tip:** feels like the tip of your nose.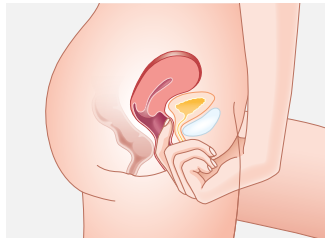

- In your vagina palpate your pubic bone.

## How to insert the Singa® diaphragm:

**NOTE:** Always insert the Singa® diaphragm before sexual intercourse.

1. Remove the Singa® diaphragm from its case.


**NOTE:** Each time before you use it, check your Singa® diaphragm for any damage.

2. Fill the membrane cup with the diaphragm gel, e. g. Caya® Gel (approx. 4 ml, roughly one teaspoonful). You may also apply some gel to the front rim to ease insertion.


Take hold of the diaphragm on the grip dimples and fold it together.


3. Find a comfortable position: lying down, squatting or with one leg elevated.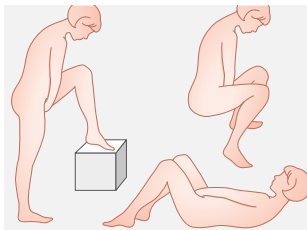


4. Hold the Singa® diaphragm with one hand. With your free hand, spread the labia. Insert the diaphragm along the back side of the vagina as deep into the vagina as you can. In the same way as you insert a tampon.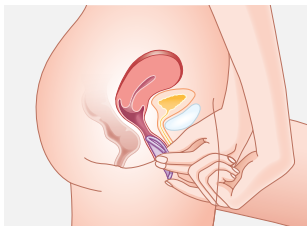

5. Now, push the front edge of the diaphragm up with your finger so as to position it in the niche behind the pubic bone.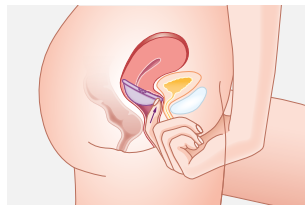

6. Use a finger to check that the cervix is covered by the membrane of the Singa® diaphragm.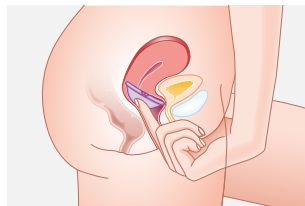

The cervix must be completely covered. You should be able to palpate it through the membrane.

**Tip:** It feels like the tip of your nose. If you are unable to feel your cervix or can feel the Singa® diaphragm (unpleasant pressure), take it out again and reinsert it.


The Singa® diaphragm is correctly positioned when the cervix is completely covered by the diaphragm and is between the back wall of the vagina and the niche in the pubic bone.

Once you have checked these two things and the Singa® diaphragm is correctly in place, you are reliably protected against unwanted pregnancy. You are now ready for intercourse.

7. The Singa® diaphragm must remain in the vagina for at least 6 hours after the last act of sexual intercourse.

**NOTE:** The Singa® diaphragm should never be left in the vagina for longer than 24 hours at a time, as this can promote the growth of bacteria inside the vagina. These bacteria can then lead to Toxic Shock Syndrome (TSS) (very rare).

8. To remove the Singa® diaphragm, hook one finger over the rim of the diaphragm and pull it downwards out of the vagina.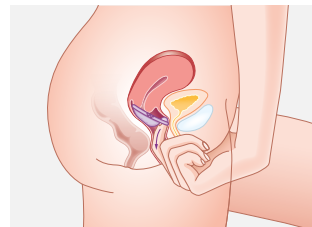

### Instructions for cleaning and care

9. After each use, the diaphragm should be carefully washed with soap and water, rinsed and then dried with a soft clean cloth.

Once you have cleaned it, check your Singa® diaphragm for any hole, tears and worn patches.

Always store the Singa® diaphragm in the case provided.

The diaphragm is for personal use only and may not be shared with others.

With proper care, the Singa® diaphragm is **reusable** for two years.


The diaphragm must be replaced straightaway if it shows any signs of damage (e. g. tears, holes).

**NOTE:** Use of this diaphragm may increase the risk of urinary tract infections or urinary tract complaints.

No contraceptive can guarantee 100 % protection against unwanted pregnancy.

Singa® diaphragm does not offer protection against sexually transmitted infections.

Always keep the Singa® diaphragm out of the sight and reach of children.

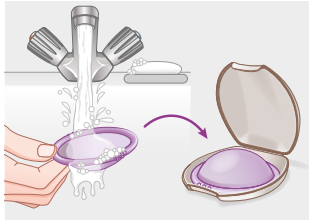**NOTE:** Never use the Singa® diaphragm together with any lubricants that contain silicone. Lubricants made of silicone oil can destroy diaphragms made of silicone rubber. For additional lubrication, use the recommended diaphragm gel or a water-based lubricant.

### When using the diaphragm, never forget the following:

- always use it together with a diaphragm gel (e. g. Caya® Gel or Contragel®)
- palpate your cervix to check if this is completely covered by the Singa® diaphragm
- leave it in the vagina for at least 6 hours after the last time you have sex
- never leave it in the vagina for longer than 24 hours at a time

### Contraindication:

- use within the first 6 weeks after delivery/birth
- severe pelvic floor weakness/uterine prolapse
- prolapse of the uterus and/or bladder
- acute or frequent urinary tract infections
- acute infections of the genital organs and/or the lesser pelvis
- if you have ever had a Toxic Shock Syndrome (very rare).

### Ask your doctor, midwife or family planning clinic for advice if:

- you are having difficulty inserting the Singa® diaphragm correctly.
- you have the impression that the diaphragm is no longer in the correct position or moves when you cough or walk.
- you are using vaginal products (suppositories or ointments) at the same time (rare).
- you have had lower abdominal/vaginal surgery.
- you have any further, personal medical questions.

### Recommendation:

- Vaginal infection: whilst being treated for a vaginal infection, you should use condoms as contraception instead of the diaphragm.
- Following a successful treatment of a vaginal infection, the Singa® diaphragm should be replaced with a new one.

### Material:

Silicone (LSR),  
Polyamide 66 (PA66)

### Contents:

- one Singa® diaphragm
- instruction manual
- storage case

## FAQs

### How safe is the Singa® diaphragm?

No contraceptive method can guarantee 100 % safety from unwanted pregnancy. The Singa® diaphragm is just as safe as the classic diaphragm. The more often you use the Singa® diaphragm, the more confident you will become in using this contraceptive.

### How do I know that Singa® is inserted correctly and fits?

Use your longest finger (usually the middle finger) to check the correct position of the diaphragm: Check whether the membrane covers the cervix completely. If this is the case, you are reliably protected.

### How long can I use the Singa® diaphragm for?

After the first time it is used, the Singa® diaphragm may be used for up to 2 years.

### Why does the Singa® diaphragm have to stay in the vagina for 6 hours after intercourse?

Sperm are no longer active after this period. These 6 hours are for your own safety and to prevent an unwanted pregnancy.

### Why should Singa® not be worn for longer than 24 hours at a time?

Wearing the Singa® diaphragm for longer than 24 hours at a time can support bacterial growth inside the vagina.

### Can I have repeated consecutive intercourse with the Singa® diaphragm?

Yes, your Singa® will remain in position, but you can apply additional diaphragm gel in front of the inserted Singa® with the aid of an applicator.

### Can the Singa® diaphragm also be used during menstruation?

In principle, this is possible.

### What do I need to do in the event of a vaginal infection?

If you have a vaginal infection (e. g. fungal infection), ask your doctor about treatment. During treatment use condoms and NOT a diaphragm. Once treatment is finished, you should replace the Singa® diaphragm with a new one.

### Can I use lubricants together with the Singa® diaphragm?

Yes, but ONLY water-based lubricant gels should, however, be used. Do not use ANY lubricant gels containing silicone together with the Singa® diaphragm! There are no studies on the tolerability of diaphragm gels when used together with commercial lubricants. Eventual limitations in terms of the efficacy of diaphragm gels cannot therefore be ruled out. The Singa® diaphragm may be used

together with condoms, as they are coated with only small amounts of silicone oil.

### Could my partner feel the Singa® diaphragm?

This is highly unlikely. However, in rare cases, your partner may feel the rim of the diaphragm.

### Does the Singa® diaphragm influence my sexual desire?

In post-marketing studies, users repeatedly report an increase in libido compared to when they use hormonal contraceptives.

### What should I do if Singa® causes pain during intercourse?

First check that Singa® diaphragm is in the correct position using a finger. If the Singa® diaphragm continues to cause pain, go and see your doctor.

### Can Singa® break or tear?

No case is known of a Singa® diaphragm becoming damaged during sexual intercourse.

### How can I positively influence the reliability of the Singa® diaphragm when using it?

It is important to use the diaphragm consistently as well as a diaphragm gel (e. g. Caya® Gel). In addition, the diaphragm should always be inserted before and not during intercourse.

### Can the Singa® diaphragm be boiled or disinfected?

After it is used, the Singa® diaphragm need only be cleaned with soap and water. Disinfectants do not need to be used and are not recommended. If you wish, you may rinse it off with boiling water. After a vaginal infection, the Singa® diaphragm should be replaced with a new one.

### Can the Singa® diaphragm continue to be used after giving birth?

No earlier than 6 weeks after giving birth, you should go and see your doctor or family planning clinic to check that it still fits correctly. Ask if you need a different size or if you can continue to use the Singa® diaphragm you have been using.

### What do I do if I have difficulty removing the Singa® diaphragm?

In rare cases, negative pressure can make the Singa® diaphragm very tight. When removing it, try adopting a different position (e. g. assume a squatting position), bear down and hook a finger over the rim of the diaphragm. If you are still unable to remove the diaphragm, ask your doctor.

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KESSEL medintim GmbH
Medical Industries Australia Hold Co Pty Ltd